



ProChem SSE

ProChem SSE is an AAMI/ISO Class 6 Chemical Emulating Indicators designed for use in steam sterilization. ProChem SSE is unique in that it is intended for use with multiple steam sterilization cycles: 3.5 minutes at 135°C (275°F) and 4 minutes at 132°C (270°F).

- **Class 6 Emulating Indicator**
- **Suitable for various cycles**
- **Purple to Green color change**
- **Cycle specific indicator**
- **Conforms to ISO 11140-1:2005**



Catalog # CI-SSIX
250 Class 6 Emulating Indicators per bag

Appropriate for the following cycles:

**3.5 Minutes
@ 275°F/135°C**

**4 Minutes
@ 270°F/132°C**

Place a Raven ProChem SSE strip in each pack, peel-pouch or tray for sterilization. Place strip at a location in the package considered most difficult for steam to reach.

When used as directed, the Raven ProChem SSE Emulating Indicator strips give visual confirmation that sterilization conditions were met at the point of placement in the sterilizer chamber. After cycle is complete, visually verify that the indicator bar has turned from purple to green. If the indicator bar is not green, it suggests inadequate sterilization conditions at point of placement.

